HEMOTRANSFER

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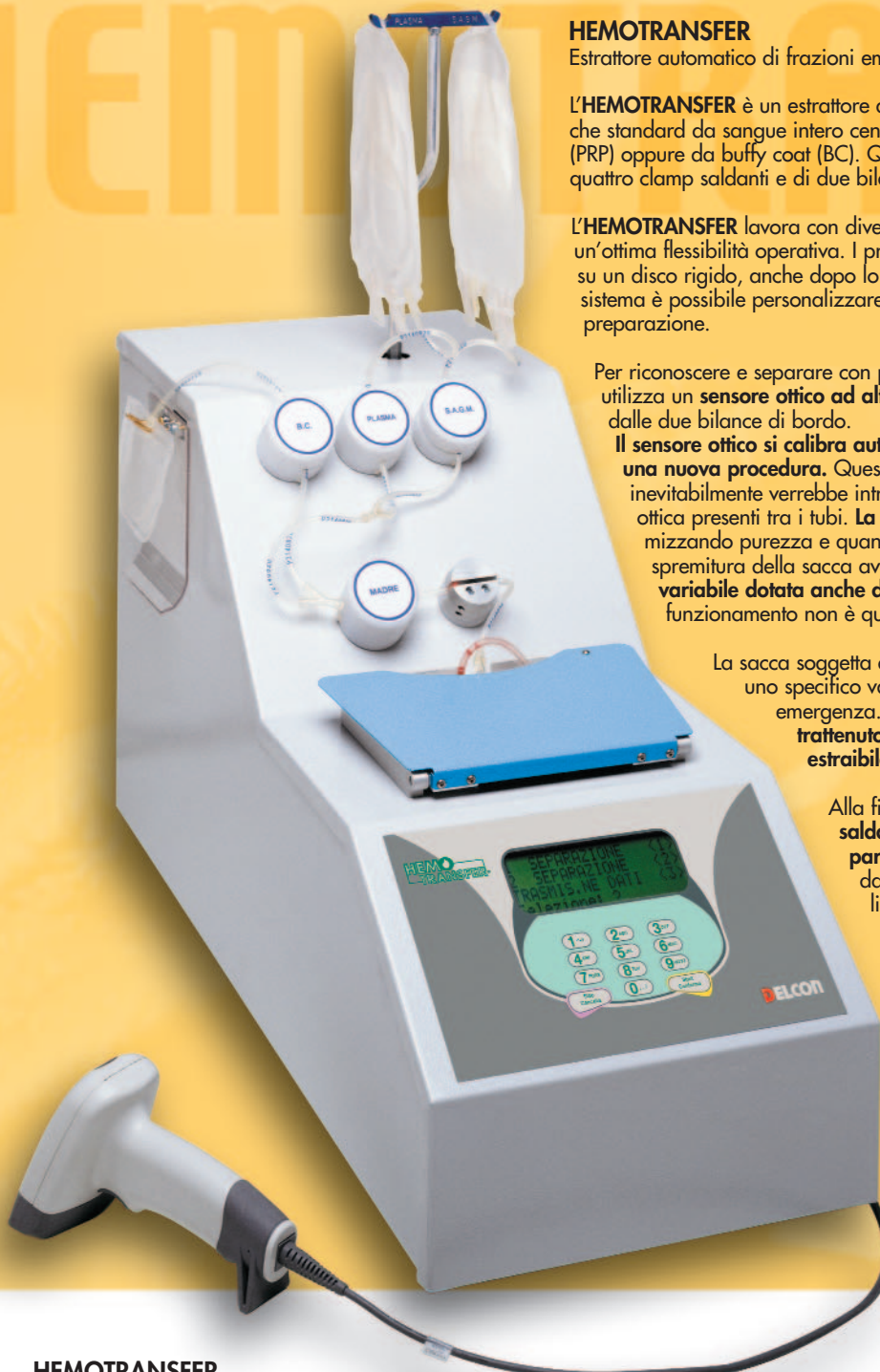
HEMOTRANSFER

Automatic blood components extractor.

HEMOTRANSFER is an automatic extractor for blood components prepared in standard bags from centrifuged whole blood or platelet rich plasma (PRP) or buffy coat. The machine is equipped with four automatic welding heads and two multi-function balances as standard.

HEMOTRANSFER works with several programs to satisfy most of routine preparations required by the modern Blood Bank. Each working protocols is stored onto a hard disk and saved even when the unit is turned off.

To recognize and properly separate the different blood components HEMOTRANSFER uses a **high efficiency optical detector** and the signal in weight from the on board balances. **The optical sensor is self calibrating to zero position (blank signal) every time a new procedure starts.** This function eliminates the background error due to the different optical transparencies among the tubes. **In addition, the sensitivity of the sensor can be adjusted** to optimize purity and yield of products. To squeeze the bag is used an electric **variable-speed press with automatic maximum thrust control**; therefore an air compressor is not necessary to run the press.

The main bag is located inside a **specific compartment equipped with a safety door and a bottom sliding tray.** In case of any bag breaking, **the liquid will not be sprayed outside the compartment falling down on the sliding shelf.**

At the end of each extraction HEMOTRANSFER can **automatically seal in sterile mode all or some of the tubing.** If required, the four welding heads can also be disabled to be used as standard clamps.

HEMOTRANSFER can also generate some records for each extraction to be transmitted to a Personal Computer.

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► **Double function balance for plasma and SAG-M**

► **Four automatic welding heads**

► **Optical sensor with adjustable sensitivity**

► **Lateral balance for BC weight**

► **Safe internal compartment for main bag**

► **Bar code reader or manual code insertion via keyboard**

Technical Features and sale code:

Code 710.101-A
HEMOTRANSFER

Main preparations: PL, RBC with SAG-M or added PL with adjustable masses, adjustable dry BC removal, PLT from PRP or from BC with adjustable weighs and HCT, function of automatic tube flushing, preparation of pediatric doses, preparations with in-line filters.

External Dimensions: 400 mm high without vertical balance rod (600 mm. with support)

260 mm wide

680 mm deep.

Net Weight: 24 Kg

Power Supply: 230V AC \pm 10% 50/60 Hz 330 VA.

Weighing Accuracy of balances: \pm 2%

Mean Extraction Time: = ~1,5 minutes for a quadruple bag.

Alarms: Acoustic and Visual

Error Codes/Alarms: Described on the display with reference codes

Available languages: Italian, English, Spanish.

Optional Accessories

Cod. 710.102

HEMOTRANSFER Software kit.

With this kit is possible to load on a PC (not included) all the data generated by n. HEMOTRANSFER.

The data are loaded on a *.txt type file, easy to be used by several applications. For the connection of more units together, please see also option code 710.103.

Cod. 710.103

CABLE NETWORK kit.

This kit is required to connect up to four units together. For connection of more than four units, please buy multiple of this kit.

Code 710.140

CCD Barcode reader.

HEMOTRANSFER